

IODOFORM POWDER



TECHNICAL & SAFETY INSTRUCTIONS FOR USE

Product Description: Iodoform powder used as dressing for treatment of extracted sockets as it is a mild analgesic which also acts as an antiseptic agent when reacting with wound secretions and fatty acids, Iodoform frees releases iodine to create a scaffold tissue by coagulating with proteinaceous substances which acts as astringent with germicidal properties.

Application : Prevention of periodontal infections, Post extraction disinfecting dressing, Disinfection of maxillary cavities.

Composition: Pure Iodoform powder.

Method of Use: Mix Iodoform powder with saline water and impregnate it on a non resorbable sterile cotton gauze. Now cut the necessary length of gauze to fill the extraction socket and pack gently into the the cavity to be treated. The gauze is non resorbable and can remain in situ for 2 to 8 day prior to removal at the end of the treatment.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes in contact with eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Storage: Store at Temp. between 5-35°C. Protect from direct sunlight. Close the container carefully after use. Keep Away from Moisture, Shelf life not affected after opening if stored in a cool & dry place.




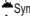
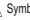








Disposal Conditions: To be disposed off based on the prevailing rules of the country in which it is used.

Disclaimer: The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.

Presentation : 1 x 15 gm Glass bottle. Ref No. ED09-01

Shelf Life: 3 years from the date of Manufacturing.

 Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/MI/MD/005962
®-Registered Trademark Only

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"  Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.